SOFTcare
SOLID

Γάντια προστασίας νιτριλίου μιας χρήσης GR
Disposable nitrile protection gloves EN
Guantes de nitrilo desechables para protección ES
Gants de protection jetables en nitrile FR
Einweg-Schutzhandschuhe aus Nitril DE

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[EN] INSTRUCTIONS FOR USE: Intended use: Protective gloves against chemical risks and microorganism are used for general purpose, food service, cleaning room and similar use. They are worn on the users' hands and fingers against chemical harm and microorganisms. **Primary material:** Nitrile synthetic rubber. **Donning:** 1. Remove all hand and wrist jewelry and wash the hands before donning. 2. Place the gloves on the prepared work surface. 3. Put a glove on your dominant hand by grabbing it with the other, remembering to only touch the inside of the gloves and slipping it over the dominant hand until it reaches finger level. 4. Use the gloved dominant hand to slip the other onto the non-dominant. Once both gloves are on, you can touch the outside of the gloves to ensure a proper fit. **Doffing:** Using the dominant hand, start by grabbing the outside of the glove on the non-dominant hand on the palm side near the cuff. 2. Pull the glove off the non-dominant hand and place it in the gloved hand, rolling it up. 3. Slip two fingers under the cuff of the other hand glove and carefully peel it off the hand without touching the wrist, turning the remaining glove inside out as it is removed and in turn encasing the first glove. 4. Gloves can be disposed. **Caution:** These gloves should only be used for the intended application and in the correct size (consult the "Size Guide"). It is recommended to check that the gloves are suitable for the intended use, as workplace conditions may differ from those of the test, in terms of temperature, friction and degradation. Also before use always check the expiry date, that are free of any defects such as holes, cracks or further faults like discolorations. Defective gloves should not be used under any circumstances. Never re-use and never share gloves with other users. Wear no longer than 2 hours. Be aware that sharp objects can puncture the gloves. Always change them in case of puncture, rip or tear. If for any reason, glove fails and chemicals come into contact with skin, consider it an exposure and seek medical attention. The information in the tables on the side of the packaging, does not reflect the actual duration of protection in the workplace and the differentiation between mixtures and pure chemicals. The chemical resistance has been assessed under laboratory conditions from samples taken from the palm only (except in cases where the glove is equal to or over 400 mm – where the cuff is tested also) and relates only to the chemical tested. It can be different if the chemical is used in a mixture. The penetration resistance has been assessed under laboratory conditions and relates only to the tested specimen. When used, protection gloves may provide less resistance to the dangerous chemical due to changes in physical properties. Movements, snagging, rubbing, degradation caused by the chemical contact etc. may reduce the actual use time significantly. For corrosive chemicals, degradation can be the most important factor to consider in selection of chemical resistant gloves. Used gloves must be disposed of after contact with chemicals according to the disposal regulations of the chemical. **Storage:** Store in a well-ventilated area, away from direct sun, extreme temperatures, fluorescent lighting and excess ozone conditions.

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EXTRA STRONG

Don't re-use

Doesn't contain natural rubber latex

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CE 2777

Personal Protective Equipment
Regulation (EU) PPER 2016/425
Category III

Suitable for food handling excluding acidic foodstuffs.
Κατάλληλο για χειρισμό τροφίμων εξαιρουμένων των οξίνων τροφίμων.
Adecuado para la manipulación de alimentos, excepto alimentos ácidos.
Convient à la manipulation des aliments, à l'exception des aliments acides.
Geeignet für den Umgang mit Lebensmitteln, ausgenommen säurehaltige Lebensmittel.

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Model Number: BS01020101041

EN ISO 21420:2020
EU Type Examination and
Ongoing Conformity by Notify Body:
NB#2777 SATRA Technology Europe Ltd.,
Bracetown Business Park,
Clonee, Dublin 15, D15 YN2P Ireland

EN ISO 374-1:2016+A1:2018 / Type B

KTP

EN ISO 374-5:2016

VIRUS Resistance against
Bacteria, Fungi & Virus: PASS

Conformity can be viewed and downloaded at
www.bournas-medicals.gr/image/DoC/Solid_O_v1.pdf
Scan to see the Declaration of conformity
Σαρώστε για να δείτε τη Δήλωση συμμόρφωσης
Escanee para ver la Declaración de conformidad
Scannez pour voir la déclaration de conformité
Scannen Sie, um die Konformitätserklärung anzuzeigen