SOFTcare ULTRA BLACK

Γάντια εξεταστικά / προστασίας νιτριλίου μιας χρήσης GR
Disposable nitrile examination / protection gloves EN
Guantes de nitrilo desechables para examen y protección ES
Gants d'examen / de protection jetables en nitrile FR
Einweg-Untersuchungs- / Schutzhandschuhe aus Nitril DE

[EN] INSTRUCTIONS FOR USE: Intended uses: (a) Medical device to prevent cross-contamination between healthcare personnel and patients during medical procedures. (b) Personal protective equipment to protect the user's hands and fingers against chemical harm and microorganisms. **Primary material:** Nitrile synthetic rubber. **Warnings:** For single use only. Not intended for use against exposure to all chemicals. Components used in making gloves may cause allergic reactions in some users, including anaphylactic responses. In case of such reaction discontinue use and consult your physician. **microorganisms.** **Donning:** 1. Remove all hand and wrist jewelry and wash the hands before donning. 2. Place the gloves on the prepared work surface. 3. Put a glove on your dominant hand by grabbing it with the other, remembering to only touch the inside of the gloves and slipping it over the dominant hand until it reaches finger level. 4. Use the gloved dominant hand to slip the other onto the non-dominant. Once both gloves are on, you can touch the outside of the gloves to ensure a proper fit. **Doffing:** 1. Using the dominant hand, start by grabbing the outside of the glove on the non-dominant hand on the palm side near the cuff. 2. Pull the glove off the non-dominant hand and place it in the gloved hand, balling it up. 3. Slip two fingers under the cuff of the other hand glove and carefully peel it off the hand without touching the wrist, turning the remaining glove inside out as it is removed and in turn encasing the first glove. 4. Gloves can be disposed. **Caution:** These gloves should only be used for the intended application and in the correct size (consult the "Size Guide"). It is recommended to check that the gloves are suitable for the intended use, as workplace conditions may differ from those of the test, in terms of temperature, friction and degradation. Also before use always check the expiry date, that are free of any defects such as holes, cracks or further faults like discolorations. Defective gloves should not be used under any circumstances. Never re-use and never share gloves with other users. Wear no longer than 2 hours. Be aware that sharp objects can puncture the gloves. Always change them in case of puncture, rip or tear. If for any reason, glove fails and chemicals come into contact with skin, consider it an exposure and seek medical attention. Don't wear the same pair for the care of more than one patient or more than one examination procedure. Always remove gloves after caring for a patient. The information in the tables on the side of the package, does not reflect the actual duration of protection in the workplace and the differentiation between mixtures and pure chemicals. The chemical resistance has been assessed under laboratory conditions from samples taken from the palm only (except in cases where the glove is equal to or over 400 mm – where the cuff is tested also) and relates only to the chemical tested. It can be different if the chemical is used in a mixture. The penetration resistance has been assessed under laboratory conditions and relates only to the tested specimen. When used, protection gloves may provide less resistance to the dangerous chemical due to changes in physical properties. Movements, snagging, rubbing, degradation caused by the chemical contact etc. may reduce the actual use time significantly. For corrosive chemicals, degradation can be the most important factor to consider in selection of chemical resistant gloves. Used gloves must be disposed of after contact with chemicals according to the disposal regulations of the chemical. **Storage:** Store in a well-ventilated area, away from direct sun, extreme temperatures, fluorescent lighting and excess ozone conditions. **[GR] ΟΔΗΓΙΕΣ ΧΡΗΣΗΣ: Προβλεπόμενες χρήσεις:** (α) Ιατροτεχνολογική συσκευή για την πρόληψη της διασταυρούμενης μόλυνσης μεταξύ του προσωπικού υγειονομικής περίθαλψης και των ασθενών κατά τη διάρκεια ιατρικών διαδικασιών. (β) Μέσο ατομικής προστασίας για την προστασία των χεριών και των δακτύλων του χρήστη από χημική βλάβη και μικροοργανισμούς. **Κυρίως υλικό:** Συνθετικό καουτσούκ νιτριλίου. **Προειδοποιήσεις:** Μόνο για μία χρήση. Δεν προορίζεται για χρήση έναντι της έκθεσης σε όλες τις χημικές ουσίες. Τα συστατικά που χρησιμοποιούνται για την κατασκευή γαντιών μπορεί να προκαλέσουν αλλεργικές αντιδράσεις σε ορισμένους χρήστες, συμπεριλαμβανομένων αναφυλακτικών αποκρίσεων. Σε περίπτωση τέτοιας αντίδρασης διακόψτε τη χρήση και συμβουλευτείτε το γιατρό σας.

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SOFTcare ULTRA BLACK

[EN] INSTRUCTIONS FOR USE: Intended uses: (a) Medical device to prevent cross-contamination between healthcare personnel and patients during medical procedures. (b) Personal protective equipment to protect the user's hands and fingers against chemical harm and microorganisms. **Primary material:** Nitrile synthetic rubber. **Warnings:** For single use only. Not intended for use against exposure to all chemicals. Components used in making gloves may cause allergic reactions in some users, including anaphylactic responses. In case of such reaction discontinue use and consult your physician, microorganisms. **Donning:** 1. Remove all hand and wrist jewelry and wash the hands before donning. 2. Place the gloves on the prepared work surface. 3. Put a glove on your dominant hand by grabbing it with the other, remembering to only touch the inside of the gloves and slipping it over the dominant hand until it reaches finger level. 4. Use the gloved dominant hand to slip the other onto the non-dominant. Once both gloves are on, you can touch the outside of the gloves to ensure a proper fit. **Doffing:** 1. Using the dominant hand, start by grabbing the outside of the glove on the non-dominant hand on the palm side near the cuff. 2. Pull the glove off the non-dominant hand and place it in the gloved hand, balling it up. 3. Slip two fingers under the cuff of the other hand glove and carefully peel it off the hand without touching the wrist, turning the remaining glove inside out as it is removed and in turn encasing the first glove. 4. Gloves can be disposed. **Caution:** These gloves should only be used for the intended application and in the correct size (consult the "Size Guide"). It is recommended to check that the gloves are suitable for the intended use, as workplace conditions may differ from those of the test, in terms of temperature, friction and degradation. Also before use always check the expiry date, that are free of any defects such as holes, cracks or further faults like discolorations. Defective gloves should not be used under any circumstances. Never re-use and never share gloves with other users. Wear no longer than 2 hours. Be aware that sharp objects can puncture the gloves. Always change them in case of puncture, rip or tear. If for any reason, glove fails and chemicals come into contact with skin, consider it an exposure and seek medical attention. Don't wear the same pair for the care of more than one patient or more than one examination procedure. Always remove gloves after caring for a patient. The information in the tables on the side of the package, does not reflect the actual duration of protection in the workplace and the differentiation between mixtures and pure chemicals. The chemical resistance has been assessed under laboratory conditions from samples taken from the palm only (except in cases where the glove is equal to or over 400 mm – where the cuff is tested also) and relates only to the chemical tested. It can be different if the chemical is used in a mixture. The penetration resistance has been assessed under laboratory conditions and relates only to the tested specimen. When used, protection gloves may provide less resistance to the dangerous chemical due to changes in physical properties. Movements, snagging, rubbing, degradation caused by the chemical contact etc. may reduce the actual use time significantly. For corrosive chemicals, degradation can be the most important factor to consider in selection of chemical resistant gloves. Used gloves must be disposed of after contact with chemicals according to the disposal regulations of the chemical. **Storage:** Store in a well-ventilated area, away from direct sun, extreme temperatures, fluorescent lighting and excess ozone conditions.

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HEAVY DUTY

Don't re-use

Doesn't contain natural rubber latex

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CE 2777

Medical Devices Regulation (EU) MDR 2017/745 Personal Protective Equipment Regulation (EU) PPER 2016/425 Category III

Suitable for food handling excluding acidic foodstuffs.
Κατάλληλο για χειρισμό τροφίμων εξαιρουμένων των όξινων τροφίμων.
Adecuado para la manipulación de alimentos, excepto alimentos ácidos.
Convient à la manipulation des aliments, à l'exception des aliments acides.
Geeignet für den Umgang mit Lebensmitteln, ausgenommen säurehaltige Lebensmittel.

EN ISO 374-1:2016+A1 2018 / Type B			EN ISO 374-4:2019
Letter Code Κωδικός Γράμματος Código de letras Code letter Buchstabencode	Chemical Permeation Χημική Διαπερατότητα Permeación química Perméation chimique Chemische Permeation	Level Επίπεδο Nivel Niveau Ebene	Degradation (%) Υποβιβασμός Degradación Dégradation Degradierung
K	40% Sodium Hydroxide	6	1.4
P	30% Hydrogen Peroxide	6	24.0
T	37% Formaldehyde	6	10.9

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Model Number: BS0102059

EN ISO 21420:2020
EU Type Examination and
Ongoing Conformity by Notify Body:
NB#2777 SATRA Technology Europe Ltd.,
Bracetown Business Park,
Clonee, Dublin 15, D15 YN2P Ireland

EN ISO 374-1:2016+A1:2018 / Type B

KTP

EN ISO 374-5:2016

VIRUS Resistance against Bacteria, Fungi & Virus: PASS

Conformity can be viewed and downloaded at
www.bournas-medicals.gr/image/Doc/Ultra_BL_v1.pdf

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