

DISINFECTION WIPE SURFACE

ONE SYSTEM BASIC



SINGLE USE WIPE DISPENSER FOR FILLING WITH DISINFECTANTS


1 / 5

Compact flexi-packaging with high stability
for particularly easy handling and fill-level control

Filled with DESOTEX **non-woven wipes**
for soaking with disinfectants

Ready to use after 15 minutes - even in areas with a increased
risk of infection - up to 60 days standing time

Flexible choice of disinfectant
based on the specific requirements

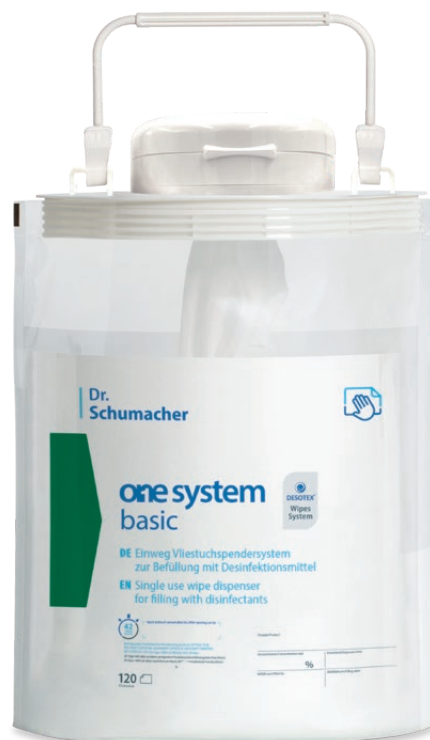

DISINFECTION WIPE SURFACE

ONE SYSTEM BASIC



PRODUCT DESCRIPTION

2 / 5

ONE SYSTEM BASIC is an innovative single-use wipe dispenser system for effective hygiene measures. In combination with suitable disinfectants, it can be used for reliable hygienic reprocessing of medical devices and medical equipment as well as all types of surface.

ONE SYSTEM BASIC is easy to use and saves you time. The system comes complete with a DESOTEX wipe roll, so it is ready to use after filling with disinfectant and allowing a 15-minute pre-impregnation time.

Together with Dr. Schumacher products, **ONE SYSTEM BASIC** can be used for a period of up to 60 days* rather than the standard 28 days. When the wipe roll runs out, the system is not reprocessed but simply emptied and disposed of.

Disinfectant	Standing time*	4-Field-Test ¹	Use also in areas with special risk of infection ²
--------------	----------------	---------------------------	---

ULTRASOL OXY	60 days	●	●
OPTISAL PLUS	42 days	●	●
BIGUANID FLÄCHE NR	42 days	●	●
CLEANISEPT	42 days	●	●
OPTISAL N	42 days	●	●
DESCOCEPT SENSITIVE	42 days	●	●

¹ – tested with all mandatory test organisms in line with EN 16615 (bactericidal/yeasticidal)

² – in accordance with information from the Association for Applied Hygiene (VAH) Disinfectant Commission in co-operation with the "4+4 Working Group": On the use of wipe dispenser systems in areas with special infection risk, Hyg Med 2014; 39 – 9.

* In comparison with other conventional surface disinfectants with a standing time of 28 days and a impregnation time of 30 minutes, the tested surface disinfectants from Dr. Schumacher achieved a presaturation time of only 15 minutes and an increased standing time of the filled system of 42/60 days.

DISINFECTION WIPE SURFACE

ONE SYSTEM BASIC



RANGE OF APPLICATION

3 / 5

Application notes

ONE SYSTEM BASIC is designed for single use and rules out any possible contamination caused by reprocessing and reuse. The system is therefore ideal for use in areas with a particularly high infection risk, as well as in all the usual applications within the healthcare sector.

The innovative single-use wipe dispenser system is simple, safe and efficient. Its design and handling are based on standard wipe dispensers, but it also offers a number of additional benefits. The opening is specifically designed for the wipe roll and allows quick access when removing the first wipe, as well as efficient soaking of the entire roll, which makes for a shorter preparation time. Filling the dispenser is intuitive and convenient for the user. The transparent pouch design means that the fill level can be checked at a glance, making it easy to re-order in good time. The dispenser features a stand-up pouch specially designed to be particularly space-saving for both storage and disposal.

The DESOTEX premium wipes in the system are made of a tear-resistant, lint-free non-woven fabric and have been comprehensively evaluated with the recommended disinfectants.

ONE SYSTEM BASIC is user-friendly, very economical and effective, ensuring optimal disinfection.

Range of application

Disposable non-woven wipe dispenser system for impregnation, hygienically storing and using wipes with suitable surface disinfectants to disinfect and clean medical devices, medical equipment and surfaces of all types.

Application

ONE SYSTEM BASIC is supplied pre-filled with suitable non-woven wipes and can be filled by the user with their choice of disinfectant:


Prepare 2,5 Liter¹ Liter ready-to-use application solution for the desired disinfectant (see the manufacturer's information for application concentrations and reaction time). Open the non-woven wipe dispenser's protective cap. Remove both parts of the seal. Fill with disinfectant solution. Thread the first wipe through the anchor-shaped opening in the dispensing element. Pull out the first wipe and discard it. Close the protective cap and attach the carrying handle if required. Complete all documentation fields on the label. The system is ready for use after 15 minutes of pre-saturation. Filling instructions are also available on request.

Notes: The extended standing time of up to 60 days applies exclusively to selected VAH-listed Dr. Schumacher surface disinfectants. In case of questions regarding the standing time and suitability of a product please contact us or the relevant manufacturer of the disinfectant.

According to the EU Medical Device Regulation, users/patients are obligated to report any serious incident that has occurred in relation to the device to the manufacturer and the competent authority of the EU Member State in which the user/patient is established.

For professional use only by personnel with corresponding specialist knowledge according to national directives.

¹Ultrasol Oxy: 2 Liter

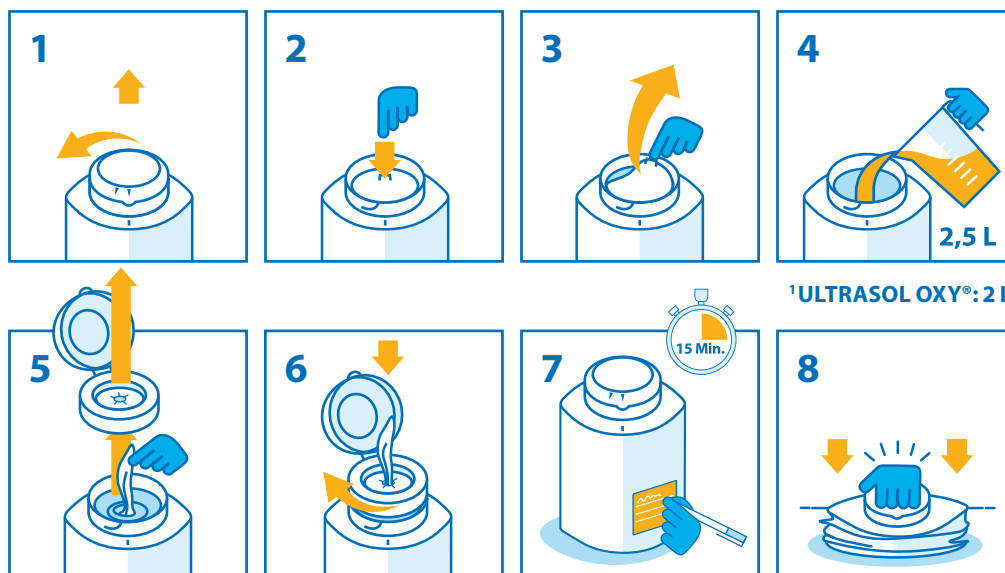

DISINFECTION WIPE SURFACE

ONE SYSTEM BASIC



PURCHASING INFORMATION

4 / 5

Product	Single unit	Delivery unit	Content	Size	REF
ONE SYSTEM BASIC	Dispenser	4	120 wipes	17,5 x 36 cm	00-915-OSEB120-01
	Wall mount	1			00-902-OSB-01

The availability of the products and container sizes depend on a successful national registration.



CERTIFICATIONS



Dr. Schumacher is certified according to DIN EN 13485, DIN EN ISO 9001, DIN EN ISO 14001, BS OHSAS 18001, has a validated environment management system according to EMAS and is a member of IHO, VCI, BAH, DGSV and of the DGKH.

DISINFECTION WIPE SURFACE

ONE SYSTEM BASIC



RECOMMENDED DISINFECTANTS

5 / 5



ONE SYSTEM BASIC



ONE SYSTEM BASIC with wall holder

FURTHER NON-WOVEN WIPE DISPENSER SYSTEMS



ECO WIPES



DESCO WIPES



ONE SYSTEM+PLUS