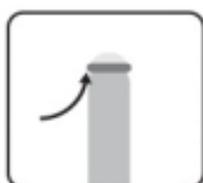
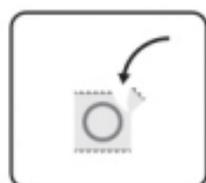
SOFTcare
e ultrasound

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EN Ultrasonic transducer probe cover

Date of issue: 01-Mar-2022 (revision 04)

Read instruction before use.

Intended users:

For healthcare professional use only.

Intended use:

As a protective cover in avoiding contamination of ultrasonic transducer probes by bodily fluids. To facilitate cleaning and sanitization of the probes.

Do not reuse as there might be an increased risk of infection.

Indication for use:

- These ultrasonic transducer probe covers (commonly known as probe cover) are regulated as an

accessory to the ultrasonic transducer probes of the Ultrasonic Imaging System, therefore it is necessary to read the probe reprocessing instructions and/or cross-referencing of probe manuals to ensure correct usage of the probe covers use with the probes.

- If the rubber material is sticky or brittle or obviously damaged, do not use it.

Instructions for use:

- Check that the expiry date is current.
- Open the probe cover package by tearing gently and carefully.
- Make sure the rolled part of the probe cover to be unrolled is on the outside.
- Place the probe cover on the head of the ultrasonic transducer probe and unroll it gently all the way to the base.
- If lubrication is required, do not use oil based lubricants with this natural latex probe cover as oil-based lubricants may damage the probe cover. Use water-based lubricants.
- For single use only.

Storage:

Store in a cool, dry place away from direct sunlight.

Disposal:

Please dispose of used probe cover hygienically and thoughtfully. Do not flush into the toilet.

Caution: This product contains natural rubber latex which might cause allergic reactions including anaphylactic shock.