

SOFTcare
LABORATORY SERIES

Ζώνες αιμοληψίας

CE MD

Tourniquet for blood collection



42 x 2,5 cm

Χρώμα: **μπλε** • Colour: **blue**

Οδηγίες χρήσης: Τοποθετείστε τη ζώνη περιμετρικά 7,5-10 εκ. πάνω από το σημείο διάτρησης & σφίξτε. Όταν το αίμα ρέει στον πρώτο σωλήνα, χαλαρώστε τη ζώνη πατώντας το κουμπί (1) για γρήγορη απελευθέρωση ή πιέζοντας την διαγραμμισμένη περιοχή (2) κάτω από το κουμπί για αργή απελευθέρωση.

Προσοχή: Μην εφαρμόζετε υψηλές πιέσεις που ενδέχεται να επηρεάσουν τη ροή του αρτηριακού αίματος στο άκρο. Απελευθερώστε τη ζώνη μόλις αρχίσει να ρέει αίμα στον πρώτο σωλήνα. Πολύ μεγάλη και πολύ υψηλή στάση μπορεί να συμβάλει στην αιμόλυση. Μην το χρησιμοποιείτε σε διαρρηγμένο δέρμα.

Instructions for use: Wrap the the tourniquet around the arm 7.5-10 cm above the puncture site and tighten it. When the blood flows into the first tube, loosen the tourniquet by pressing the button (1) for quick release or by pressing the lined area (2) below the button (slow release).

Caution: Do not apply high pressures that may impair arterial blood flow to the extremity. Release tourniquet as soon as blood starts to flow into the first tube. Too long and too high stasis may contribute to haemolysis. Do not use on broken skin.

